


## Setting up your Nicovape<sup>®</sup> Q


Your Nicovape<sup>®</sup> Q system consists of a **Nicovape<sup>®</sup> Q Device** (battery) and **Nicovape<sup>®</sup> Q Cartridges**.

With no buttons or settings, it will activate automatically with each puff, and enter standby mode when not in use.


1. Remove a new Nicovape<sup>®</sup> Q Cartridge from its blister packaging and remove the top and bottom silicone plugs.

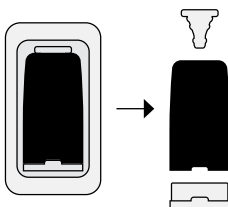

2. Place your new Nicovape<sup>®</sup> Q Cartridge on top of your Nicovape<sup>®</sup> Q Device, then push down firmly to pair them.

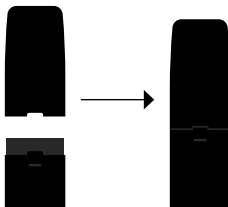

3. Your Nicovape<sup>®</sup> Q Device will vibrate and the LED will show green, indicating that it has successfully paired with a genuine Nicovape<sup>®</sup> Q Cartridge.

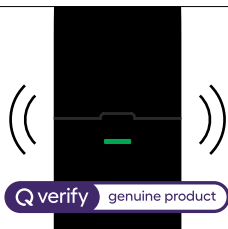

**Your Nicovape<sup>®</sup> Q is now ready to use.**

## mynicovape<sup>®</sup> Q

Tap your Nicovape<sup>®</sup> Q device on any smartphone to access your own **mynicovapeQ** portal.



For more information, scan the QR code or visit [nicovapeq.com/my](https://nicovapeq.com/my)



## Using your Nicovape® Q

Your Nicovape® Q is activated as soon as you take a puff through the mouthpiece – you will feel a gentle vibration while inhaling.

- 1. Draw firmly through the mouthpiece for around two seconds. Aim to fill your mouth with air through the Nicovape® Q Device.**
- 2. Remove the Nicovape® Q Device from your mouth and inhale deeply, followed by a slow exhalation.**

Wait at least 20 seconds before taking another puff of Nicovape® Q. This will allow you to determine if you have satisfied your cravings.

**Important:** To ensure that a cartridge lasts you a full day, limit the number of two-second puffs you take to no more than 20 per hour. A cartridge will deliver approximately 300 two-second puffs.

## Charging your Nicovape® Q

When your Nicovape® Q shows an amber or red light, use the USB-C charging cable supplied to connect it to a charger. You can plug the charging cable into any USB port or a wall charger (max 5v).

Charging your Nicovape® Q from empty to 100% will take about 90 minutes. A fully charged battery contains enough power to consume a whole Nicovape® Q Cartridge.

**Important:** Ensure the LED shows a pulsing green light when charging. The LED will turn off when the battery is fully charged.

## What you need to know about consuming nicotine

Once a cigarette is lit, you are forced to take 10-12 puffs until it is finished.

The Nicovape® Q system delivers nicotine quickly and consistently, just like a cigarette, but without ‘forcing’ you to take all your puffs in one go. It can be used only when you feel the urge to smoke, two or three puffs at a time.

**While nicotine is addictive, it is not the cause of smoking-related diseases.** 250 of the 7,000 chemicals in tobacco smoke are known to cause serious harm. Almost all of these harmful chemicals are not found in the Nicovape® Q system’s aerosol. Where they are found, they are at levels that do not pose a significant health risk.

**We’re always here to help.**

Scan the QR code or visit [nicovapeq.com/help](https://nicovapeq.com/help) to access our Help Centre and live support



Nicovape Q products are TGA notified therapeutic vaping goods indicated for smoking cessation or the management of nicotine dependence. TGA Notification ID numbers: VG-2024-NTF-00234, VG-2024-NTF-00241, VG-2024-NTF-00240, VG-2024-NTF-00235, VG-2024-NTF-00242, VG-2024-NTF-00239, VG-2024-NTF-00237, VG-2024-NTF-00243, VG-2024-NTF-00238, VG-2024-NTF-00236.

© 2024 Liber Pharmaceuticals Ltd. NICOVAPE® is a registered trademark

AU-NVQ-PT-24001-1